

The NEW SacroLoc®

PROVEN EFFECTIVENESS WITH IMPROVED COMFORT

BAUERFEIND.COM


MODE OF ACTION

SacroLoc stabilizes the pelvis and relieves the sacroiliac joints. The circular compression of the elastic net material and two tensioning straps induces a strengthening of the pelvis and relieves the local ligaments. The intensity of the effect can be regulated individually via the tensioning straps. Two dorsal pads are positioned precisely on the SIJ via the patented strap system. In movement, they exert a massage effect and enhance the relieving function of the orthosis. The massage nubs act targetedly on triggerpoints as well as on the edges of the stabilizing ligaments, tendons and muscles. This promotes blood circulation, reduces pain and relieves tension in the lumbosacral area.

- **targetedly reduces pain**
via two massage pads on the sacroiliac joints, that can be positioned precisely
- **precision fit**
due to elastic net material and anatomical fit
- **relieves immediately**
due to circular compression through individually adaptable tensioning straps
- **easy donning**
due to practical velcro with handloops
- **unique comfort**
through breathable net materials and slim, flat closing with soft edges

INDICATIONS

- Diseases of the sacroiliac joint (SI joint syndrome, SI joint arthrosis, SI joint instability)
- Myalgia and tendinopathy in the pelvic region
- Pelvic girdle instability (e.g. symphysis rupture and relaxation)
- Sacroiliac joint blockage (acute and prophylactic)
- Structural disturbances following spondylodeses in the transition from L5 to S1
- Conservative treatment following pelvic fractures

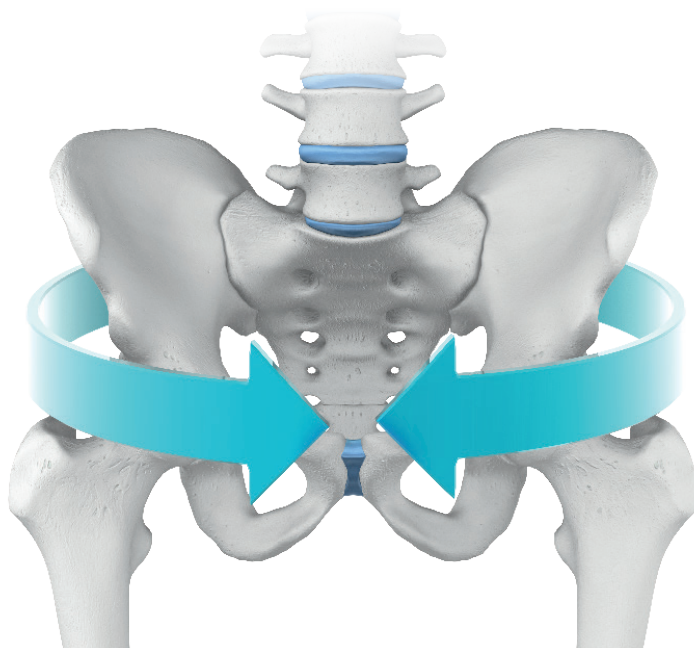

INNOVATIVE CHARACTER

- **hip support of light, elastic net material**
high wearing comfort especially through breathability, precision fit and position stability
- **straps of monofilament with patented route**
breathable straps dorsally redirected from the outside to the inside of the hip support, pads are fixed directly on the inner part of the straps, pads are controlled directly

The combination of new bandage and new strap route results in a proven effectiveness:

- relieve of ligaments (esp. lig. sacrotuberale & sacrospinale)
- limitation of pelvic nutation